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DENTAL MERCURY

INSTRUCTION FOR USE:

Dental mercury is liquid mercury intended for dental amalgams. For single use.

Indication:

Use according to amalgamator manufacturer instructions and according to dental alloy powder producer instructions.

Fillings for dental restorations.

Contraindication:

In case of allergic reaction to mercury.

For children under six years of age.

For pregnant or nursing women.

Avoid contact between dental amalgam and other metal materials.

ACCORDING TO ISO 24234

Delivered amount (g): 250, 500

Contains: Hg (liquid)

Storage conditions: at temperature no higher than 28°C, in clean surroundings without chemical vapours.

Shelf life: when stored properly until expiring date.

Specified application purpose: Dental mercury is used for preparing dental amalgam as tooth filling after mixing with dental amalgam alloy.

Processing instruction:

Use according to according to dental alloy powder producer instructions.

CAUTION
This product
contains mercury

Precautionary notes:

Properties changes:

Do not use dental amalgam in case of incorrect consistency of amalgam.

Effect of mercury on metals: Mercury corrodes and embrittles particular metals and their alloys. Avoid unnecessary contact between mercury and those metals.

Spillages: Mercury presents health hazard if handled incorrectly. Mercury is toxic by vapour inhalation and the effect is cumulative. Spillages of mercury should be removed immediately, including from places which are difficult to access. Use a plastic syringe to draw it up. Smaller quantities can be covered by sulfur powder and removed. Individual small droplets can be picked up by tin foil (Sn) and removed. Avoid inhalation of the vapour. During and after clean-up thoroughly ventilate the area where the spill has occurred.

Moisture contamination: If moisture is introduced into the amalgam before it has set, properties such as strength and corrosion resistance can be affected adversely. If the alloy contains zinc, such contamination can result in an excessive (delayed expansion). Use a dry field, whenever possible.

Waste disposal:

Waste materials and all primary containers that have held mercury shall be disposed of following appropriate management practice. Used primary containers and waste materials have to be handled as the very dangerous wastes - they contain remainders of dangerous matters – mercury.

Producer:

SAFINA, a.s., Vídeňská 104,
Vestec, PC: 252 42, Czech Republic
VAT: CZ03214257, www.safina.cz

Valid from October 1st, 2015

PRECIOUS METALS PROCESSING AND RECYCLING